Freedom CPAP Battery 160 – Quick Guide

Product Overview

The Freedom CPAP Battery 160 is a portable lithium-ion battery designed to power CPAP machines and other small electronic devices when AC power is unavailable. It supports backup power mode, dual battery expansion, and USB charging for mobile devices.

Technical Specifications

Battery Type: Rechargeable Lithium-ion (LG/Samsung ICR18650)
Capacity: 160Wh
Output Voltage: 12V-24V DC (auto-adjusted via DC output cable)
Max Output Current: 8A
USB Output: QC 3.0 (5V / 2.4A max)
Charging Input: 10V-28V DC
Charge Time: ~3-4 hours (per battery)
Battery Life: ~500 full charge cycles (~2-3 years)
Operating Temperature: 23°F to 104°F (-5°C to 40°C)
Storage Temperature: -4°F to 104°F (-20°C to 40°C)
Weight: 1.7 lbs (0.77 kg)

Battery Level Indicator (LEDs)

- 5 lights: 81%-100%
- 4 lights: 61%-80%
- 3 lights: 41%-60%
- 2 lights: 21%-40%
- 1 light: 1%-20%

Button & Port Functions

- Power Button: The battery turns ON/OFF automatically when a device is connected/disconnected.
- Charge Level/Reset Button: Press to check battery status.
- DC OUT Port: Connects to CPAP machine (12V or 24V, based on cable).
- USB OUT Port: Charges small devices like phones and tablets.
- DC IN Port: Used for AC charging.

Key Features

- ✓ FAA-compliant (carry-on only for air travel)
- ✓ Power backup mode - battery automatically takes over during power outages
- ✓ Dual battery support - extend CPAP runtime by bridging two batteries
- ✓ Overcharge, short circuit, and temperature protection
- ✓ Optimized for CPAP machines, but also supports other devices

Important Usage Notes

- ⚠ Heated humidifiers & heated tubing significantly reduce runtime (disable for longer usage).
- ⚠ Disconnect all devices when not in use to allow complete battery shutdown.
- ⚠ If battery stops working immediately, check if the CPAP requires more than 8A or a constant voltage output.

EC REP

OBELIS S. A., ADDRESS: BD. GENERAL
WAHIS, 53, 1030 BRUSSELS, BELGIUM.

MANUFACTURED BY:
JAVID DE MEXICO S. DE R.L.
DE C.V. (RESPLABS)
BLVD LUIS DONALDO COLOSIO
2850-D PARQUE INDUSTRIAL DE
NOGALES, NOGALES,
SONORA 84094 MEXICO

Made in Mexico.

Distributed by: cphero inc.
2015 N Forbes Blvd, Suite 101, Tucson, AZ 85745, USA
cphero.com

For full manual
& support, scan
the QR code

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EC REP

OBELIS S. A., ADDRESS: BD. GENERAL
WANIS, 53, 1030 BRUSSELS, BELGIUM.

FABRICADO POR:
JAVID DE MEXICO S. DE R.L.
DE C.V. (RESPLABS)
BLVD LUIS DONALDO COLOSIO
2850-0 PARQUE INDUSTRIAL DE
NOGALES, NOGALES,
SONORA 84094 MEXICO

Distribuido por: caphero inc.
2015 N Forbes Blvd, Suite 101, Tucson, AZ 85745, USA
caphero.com

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